


OnePacs Peer Review Guide


-  Check the Peer Review tab at the beginning of every shift to see if there are cases that need to be reviewed before the shift or during shift downtime.

- 1 Navigate to <https://concord.onepacs.com/views/worklist/WorklistQuery.action>


- 2 Click "Peer Review" worklist tab

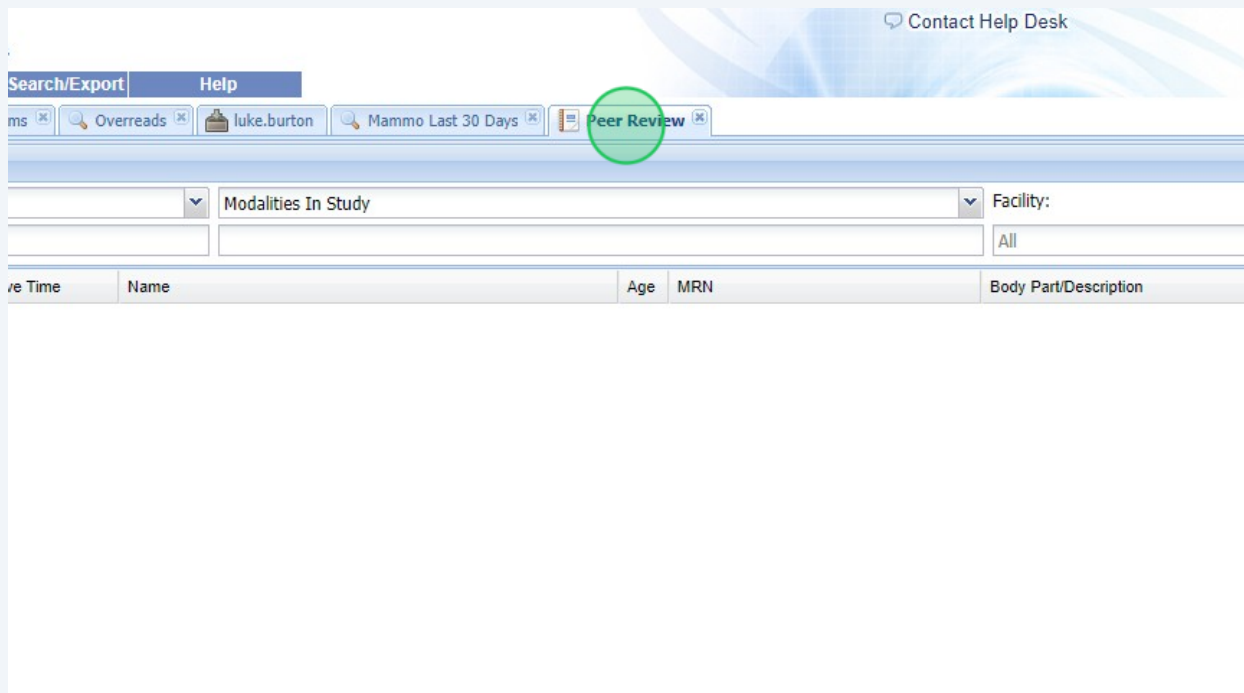

3 Click the last icon in the toolbar column, 'Open peer review and view case'.

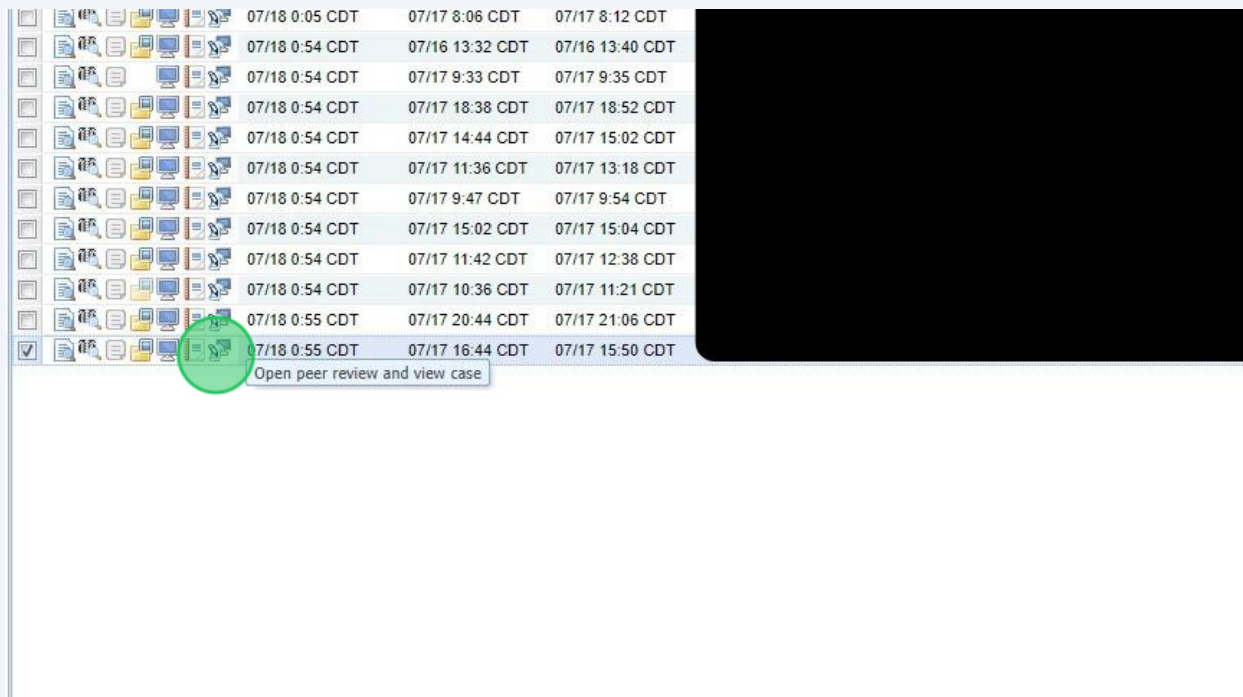

4 A window will pop up that will have the radiologist's report and allow you to complete the peer review.

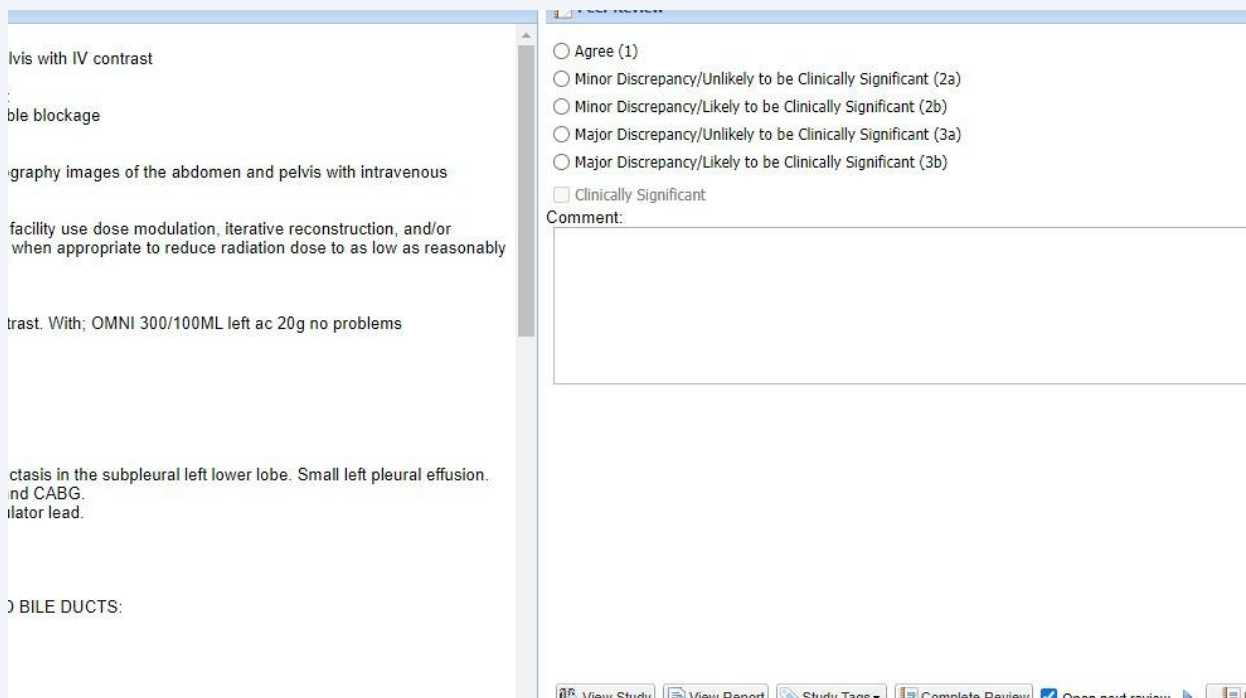

5

Once you have selected a level of agreement and added any pertinent comments, click 'Complete Review'. If 'Open next review' is checked, it will automatically pull up the next case to peer review.

The screenshot displays a medical review interface. On the left, a vertical list of studies is visible, with the text "problems" and "ll left pleural effusion." appearing. The main area on the right is a large, empty rectangular box. At the bottom, a toolbar contains several buttons: "View Study", "View Report", "Study Tags", "Complete Review" (highlighted with a green circle), "Open next review" (with a checked checkbox), "Delete", and "Cancel". Below the toolbar, a table lists studies with icons and text descriptions.

ent study)		
		right flank pain possible blockage (Hx) / abd pain rt flank possible blockage (DICOM Hx)
		injury, trauma injury, trauma (Hx) / STEPT OFF MOTORCYCLE AND FELL BACKWARDS (DICOM Hx)
		injury, trauma injury, trauma (Hx) / STEPPED OFF MOTORCYCLE AND FELL BACKWARDS (DICOM Hx)



If you are experiencing issues, reach out to the main line (available 24/7) for help at 469-677-7430.